

Molecular Testing for: Interstitial Cystitis • Recurrent, Persistent, or Complicated UTI • Prostatitis • Simple Cystitis

TEST	INTERSTITIAL CYSTITIS, RECURRENT, PERSISTENT, OR COMPLICATED UTI, AND SIMPLE CYSTITIS	PROSTATITIS
SPECIMEN TYPE	Urine	Urine after Prostatic Massage
ACCEPTABLE SPECIMENS	Voided (Midstream) Urine; Catheter Urine	Voided (First stream) Urine
SPECIMEN COLLECTION	Urine in 4mL Grey Top Tube	Urine in 4mL Grey Top Tube
SPECIMEN STORAGE	Room Temperature	Room Temperature
SPECIMEN TRANSPORTATION	FedEx Priority* Overnight	FedEx Priority* Overnight
REJECTION CRITERIA	- 5 Days Collection	- 5 Days Collection
	- Frozen Specimens	- Frozen Specimens
	- Foley Catheter Tips	- Foley Catheter Tips
		- Prostrate Massage Fluid Only
		-Semen
TURNAROUND TIME (TAT)	24 - 48 Hours of Specimen Receipt at Pathnostics	24 - 48 Hours of Specimen Receipt at Pathnostics

*Please consider including an ice pack in your shipments during warmer weather
(This Test was developed and its performance characteristics were determined by Pathnostics)

PLEASE NOTE:

- Use sterile technique
- Label all specimen with 2 patient identifiers (Acceptable Identifiers include but not limited to Patient name, Date of Birth, Requisition Number, Collection Date)
- Enclose all necessary paperwork
- Samples are accepted Monday-Saturday, 8am-6pm PST



QUESTIONS?

Contact the Pathnostics Support Team at:

☎ 800.493.4490 ✉ customer@pathnostics.com